




Product Data Sheet

2401

Rev 2
Effective date: 4 Feb 2011
Page 1 of 1
Authorisation: CO 2054

Product Description The Formaldehyde control indicator is designed for use as an (LTSF) indicator for low temperature steam formaldehyde sterilization. Upon achieving successful cycle conditions the indicator will turn from blue to green. It is designed to be used adjacent to instruments being sterilized inside a pack, bag or pouch. The primary case is printed with lot number and expiry date.

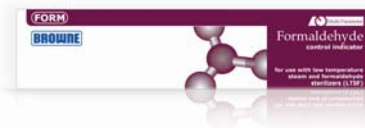

Specification When processed under conditions of low temperature steam and formaldehyde (LTSF) sterilization processes, the indicator will turn from blue to green

Applicable standards ISO 11140-1:2005 Class 4

Product Size 144mm x 22mm

Packaging Primary case contents: 100 indicators Size: 164mm x 36mm x 30mm Weight: 100g Gross

Secondary case contents: 50 cases of 100 indicators Size: 400mm x 180mm x 180mm Weight: 3.5kg Gross

Recommended Storage Cool, dark, dry conditions before and after use: 0 - 30°C 30 – 60 % RH

Shelf Life 24 months from date of manufacture

Active Components Each indicator, which is the indicating medium, contains 0.7mg dyes and 0.7mg reagents. None of the substances used in the formulation are known carcinogens, nor do they contain any heavy metals.

The component chemicals are potentially irritant. Being fully encapsulated, this should present no occasion for contact with the chemical components. If however, contact is made due to breakdown of the encapsulation, wash the skin with soap and water.

If ingested wash out the mouth thoroughly and give plenty of water to drink.

Safe Usage The indicator and packaging contain no added heavy metals, no known carcinogens, and no added rubber latex. The indicator is considered safe when used under normal conditions.

When used as intended, the indicator does not release any substances known to be toxic in sufficient quantities to cause a health hazard, during or after the sterilization process for which it is designated, in accordance with ISO 11140-1:2005, 5.9

Disposal Treat unwanted indicators and packaging in the same way as normal paper waste.

