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
MELAcontrol/PRO is a sterilization indicator and batch monitoring system that consists of a Helix test body and an indicator test strip.

MELAcontrol/PRO is only designed for testing Class B autoclaves, i.e. autoclaves which work with the fractionated pre- vacuum method.


1. Unscrew the cap from the helix test body.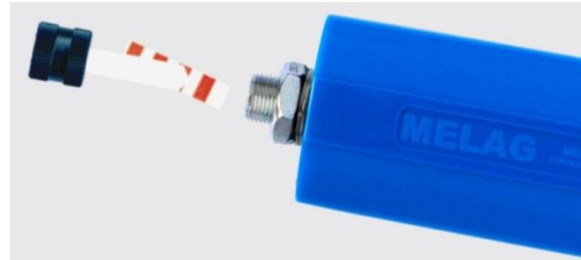

2. Insert an indicator test strip into the aperture of the plastic holder in the end cap.

3. Screw the cap in the helix test body again.

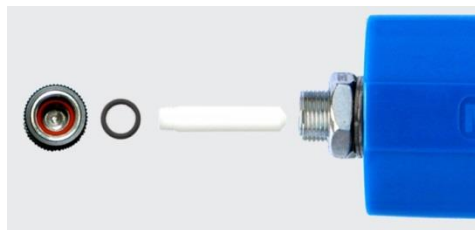It is recommended to change the black sealing that lays inside the cap after 250 tests. A new sealing is included in every refill package of indicator test strips.

4. To renew the sealing unscrew the white plastic holder out of the end cap. Take the sealing ring out with a tweezers and replace it by a new one.

**DANGER**

If the Helix test body does not lay horizontally during the sterilization, scalding condensate can flow out and can lead to scalds.

**IMPORTANT!**

Before each test you have to insert a new unused indicator strip into the helix test body. Let the helix test body dry before reuse. It is useful to blow the hose and the holding fixture of the indicator strip with compressed air. You can order the refill package with 250 indicator test strips and a red sealing ring under the MELAG order number 01076.

**Use as working-day routine test of a Class B autoclave**

1. Place the test body with the indicator strip inserted on a tray into the empty chamber.
2. Start the autoclave with the **Universal-Program**.

**Use as batch control system**

1. Include the test body to the items to be sterilized and place it into the autoclave.
2. Then start the respective program (B-cycle, 134°C, regarding DIN EN 13060).

After taking the MELAcontrol/PRO out of the autoclave, unscrew the end cap and check the indicator strip. If it shows a complete colour change it can be assumed that the autoclave sterilizes successfully.

**Evaluation**

Light brown discoloration

Temperature, no air removal, consequently no steam penetration

Incomplete change of colour:

Insufficient air removal, insufficient steam penetration

Complete change of colour:

Sufficient steam penetration, sterilization temperature and sterilization time.

**HINT!**

Residuals in the steam, e.g. oil, can lead to discoloration of the Helix test body. Those discolorations can be removed by cleaning the test body with a common household cleaner. This is no reason for complaint.