SARAYA

Manual Dispenser

MD-450

For indoor use

English

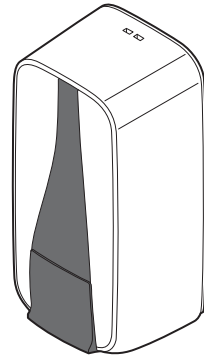
Instruction manual

Thank you very much for purchasing the Saraya MD-450 Manual Dispenser.

You are urged to thoroughly read this instruction manual before operating the MD-450.

Keep this instruction manual handy for future reference.

For further information, please contact your sales representative.

Package contents

•MD-450	1	•Screw set	
•Key	1	tapping screws.....	3
•Wet sheet for cleaning"①"	1	•plugs	3
•Double sided tape"②"	1	•Instruction manual	

Tools for installation

- Electric drill ●Drill bit(6mm,2.5mm) ●Philips screwdriver
- Hammer (resin head) ●Tape measure, ruler

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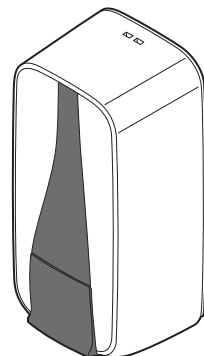
1 Safety precautions

You are recommended to read these safety precautions before operating the unit. Because these precautions are designed to protect you and your surroundings, always follow the instructions.

Safety precautions are classified as follows.

	Warning	Improper handling may result in death or serious injury.
	Caution	Improper handling may result in injury and/ or property damage.
Symbols		Strictly prohibited
		Example
		Strictly observe the instructions
		Example

Installation precautions	
	Warning
	Install the unit within its standard specifications. Failure in doing so may result in injury or breakdown.
	Install the unit on a sturdy, flat surface. Failure in doing so may result in damage to the unit or injury.
	Do not install near sources of open flame. Doing so may result in deformation or fire.
	Do not install in an area where the unit is exposed to the direct sunlight or high temperatures. Doing so may result in fire, deformation or malfunction.
	Install the unit in an area that can be ventilated well during use. Failure in doing so may result in irritation or adverse health effects.

Installation precautions	
	Caution
	Install the unit in an area free of moisture or dust. Failure in doing so may result in malfunction.
	Install the unit in an area free of condensation. Failure in doing so may result in malfunction.

Usage precautions	
	Warning
	No one other than authorized technician should attempt to disassemble, repair or modify the unit. The unit may malfunction. Contact your sales representative if you have any questions.
	Use the unit within its standard specifications. Failure in doing so may result in injury or malfunction.
	Do not smoke or light a candle near the unit. Doing so may result in fire, deformation or malfunction.
	Do not get the unit wet. Failure in doing so may result in malfunction.
	Thoroughly read the precautions for chemicals used with the unit. Failure in doing so may result in injury.

Usage precautions	
	Caution
	Do not place objects on the unit. Doing so may result in the unit falling and injury.
	Do not drop or subject to strong impact. Abusive handling can result in damaging the unit.
	Do not open and close the front cover roughly. Doing so may result in damaging the unit.
	Following the laws and regulations of your country or area when disposing of the unit.
	Be careful not to get your finger caught when opening and closing the cover. Failure in doing so may result in injury.
	Give the unit regular maintenance. For safety and good performance, regular maintenance and cleaning are recommended.
	Before long periods of non-use or storage, take out the bottle and clean the unit. Not doing so may cause malfunction or deterioration in the quality of chemicals.

	Use only chemicals approved by Saraya. Failure in doing so may cause unexpected accident or damage to the unit.
	Do not allow the solution to spill when installing the bottle. Wipe up any spills immediately. Failure to do so may cause the floor to become slippery and dangerous. The solution may cause the floor to change colors.
	Remove the bottle when moving the unit. Failure in doing so may cause a liquid spill.

SARAYA CO., LTD 2-2-8 Yuzato, Higashimiyoshi-ku Osaka
Global Operations Division 546-0013 Japan

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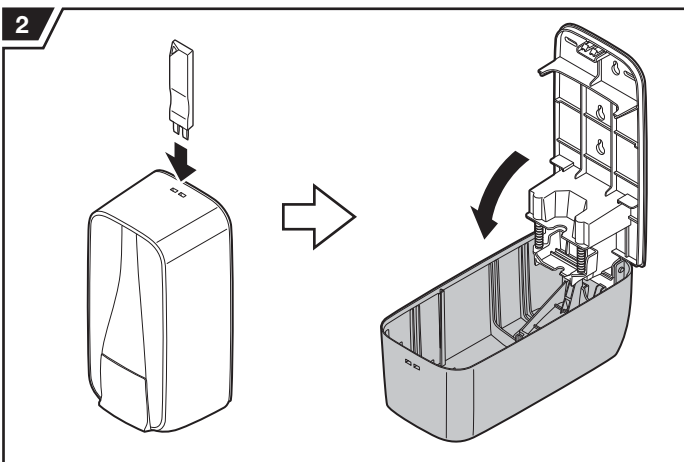
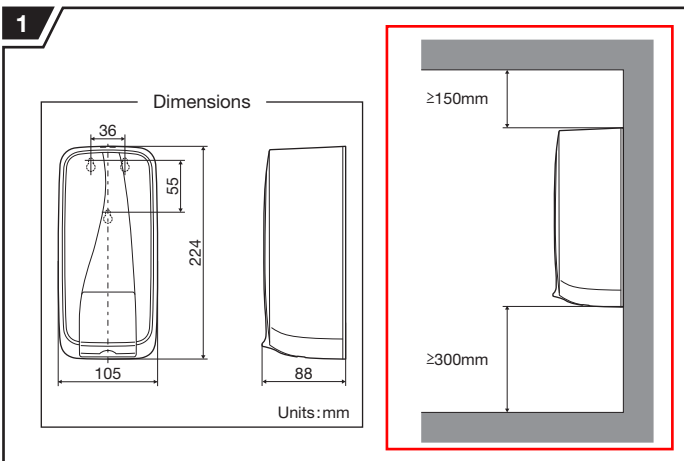
2

3

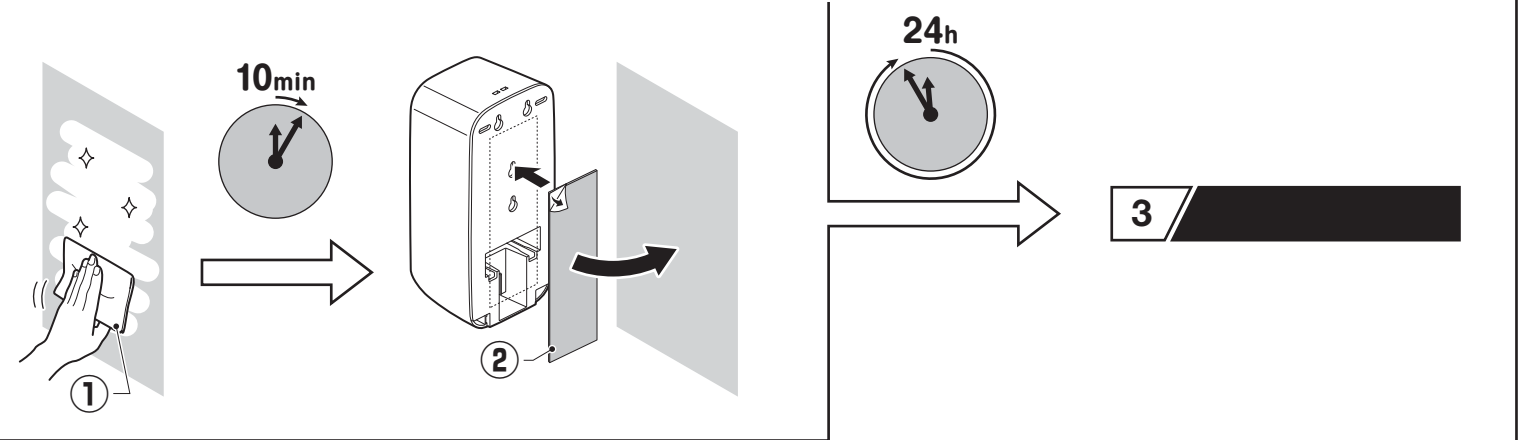
EN: ■ Installation with double sided tape
Do not mount the unit on a painted, wallpapered or rough surface.

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
EN: ■ Installing with screws
Do not mount the unit on a rough surface.

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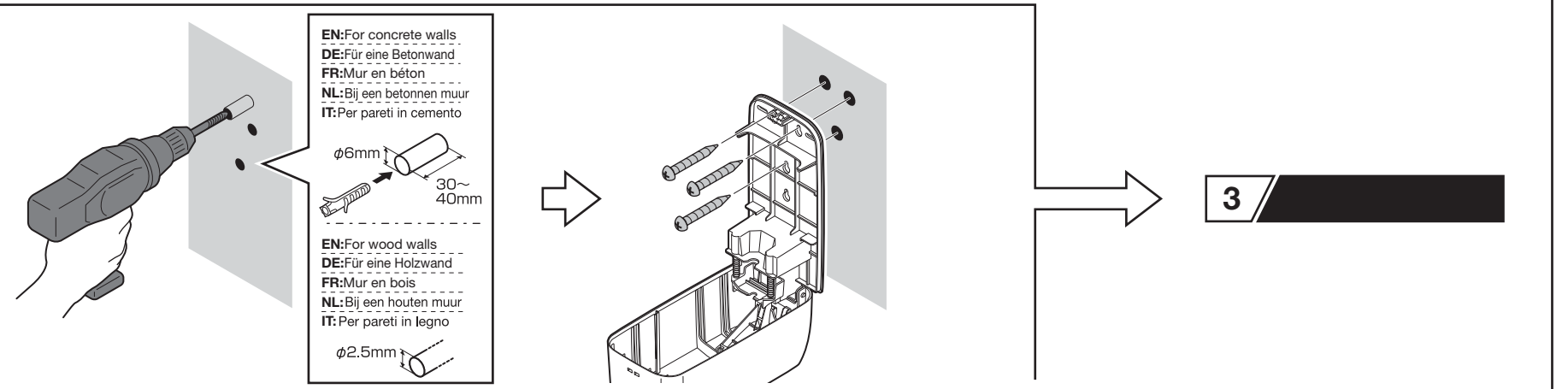
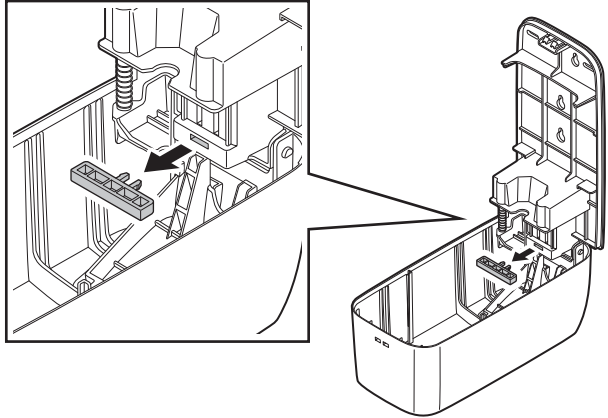
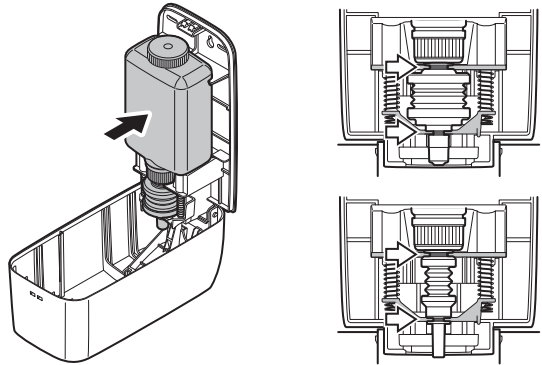
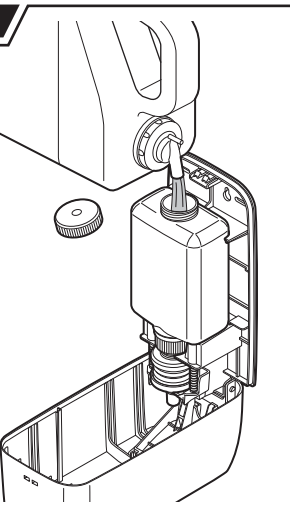
4

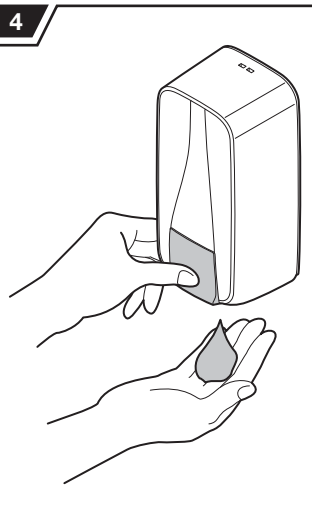4

EN:
Any dirty areas should be cleaned with a dry, soft cloth. Heavy stains should be cleaned with neutral detergent and water, wiped with a moist towel, and dried.

*Do not use thinner, benzene, petroleum, powder soap, polishing powder, non-neutral detergent, boiling water, acids, alkaline materials, brushes, etc. to clean the surface. Follow the instructions when using a chemically pre-treated cloth.

If the nozzle becomes clogged, it may cause incorrect dispensing or malfunction.

1. Soak the end of the nozzle in hot water (approx. 70-80°C) for 2 minutes then check if the pump is dispensing properly.
2. This pump will suffer from daily wear and tear. If the dispensing quality is not acceptable, please replace with a new pump bottle.

*Please contact your Saraya sales representative for more information on replacement pump bottles.

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5

EN: Investigate the following when a failure is suspected.

Problem	Checking point	Approach	Page
When chemicals is not dispensing.	Life of chemicals level	Replace the chemicals to a new one	3
	The bottle is not properly installed	Install the bottle correctly	
	The nozzle is clogged	Clean the nozzle	4

Investigate the points above first. If the unit does not operate normally, contact your sales representative and do not attempt to repair the unit by yourself.

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6

Item	Details
Name / Model	Manual Dispenser / MD-450
Dimensions	W105 x D88 x H224 mm
Weight	Approx. 310g (excluding packaging, accessories, and chemicals)
Material	Case : ABS Lever, Slide : POM Spacer : PP
Operational temperature / humidity	5-40°C / 20-85%(no condensation)
Chemicals / Capacity	Only chemicals indicated by Saraya. 450ml

※Specifications are subject to change without prior notice for improvement.

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