steamer X3

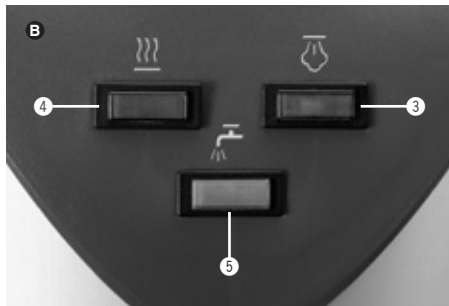
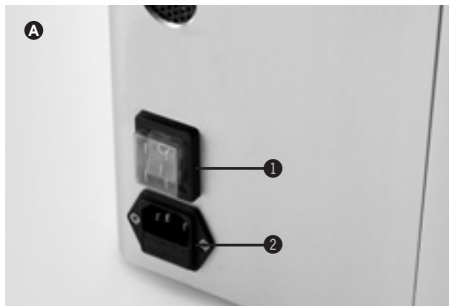
Betriebsanleitung
Operating instructions
Mode de fonctionnement
Istruzioni sul funzionamento
Instrucciones de servicio

Deutsch	04 - 09
English	10 - 15
Français	16 - 21
Italiano	22 - 27
Español	28 - 33

STEAMER X3

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STEAMER X3

Translation of the original operating instructions

- ① Power switch
- ② Electrical power cord connection with ground lead

Indicator lamp:

- ③ Green: „Device ready for use“
- ④ Red: „Heating in progress“
- ⑤ Orange: „Minimum water level reached - turn off machine and refill.“
- ⑥ Safety cap and filler neck
- ⑦ Holder for steam head
- ⑧ Steam head
- ⑨ Drainage screw
- ⑩ Gasket in safety cap

A stainless steel tank with a capacity of 3.7l is integrated in the Steamer X3. The housing is made of aluminium. The housing surface is smooth for easy cleaning. Steamer X3 can be mounted anywhere, independent of where the water hookup is located. The cleaning system (140 °C/4 bar) is always effective. When the minimum water level has been reached, a warning light will turn on and the heating system will automatically shut down. Thanks to the structure of the boiler, saturated steam will remain available for a certain amount of time even after the heating system has shut down.

UNPACKING AND CHECKING

Once the package has been opened, check that the instrument has not been damaged during transport. A complaint must be immediately lodged with the supplier if any damage is found. The instrument is essentially ready-for-use upon arrival. Check the voltage indications and make sure that the voltage rating listed on the tag on the back of the housing coincides with the line voltage of the location where the instrument will be used.

PRODUCT DESCRIPTION

Steamer X3 is a high quality, universal steam cleaner for use in the following applications:

- _ cleaning of models, dies, metallic structures, a wide range of small objects, ceramics or plastic prosthetics
- _ removing of waxy residues
- _ cleaning of articulators
- _ cleaning of dental tools and instruments
- _ cleaning of jewelry

INSTRUMENT INSTALLATION OR EVENTUAL WALL MOUNTING

Steamer X3 is designed to be used as either a free-standing or a wall mounted device. If used as a free-standing device, it is recommended that it be placed on a stable surface. Due to the production of heat and the convector system for the dispersion of heat, the Steamer X3

should not be stored in a closet. To use Steamer X3 as a wall instrument, the instrument must be mounted to the desired wall.

A mounting template has been provided which can be used to mark the drilling points on the wall in a quick and precise manner. For wall mounting, check beforehand that the selected wall can support the weight of the instrument and that there is sufficient space above it to fill the boiler.

WALL MOUNTING THE INSTRUMENT

- _ Firstly check that the wall chosen for the mounting of the Steamer X3 is stable and strong enough to sustain its weight (no plaster walls, etc.). **WARNING:** Special toggle bolts must be used for plasterboard walls!
- _ Place the mounting template in the desired position. **WARNING:** Leave enough space above the instrument so that the boiler can be filled. Make sure that the holes are horizontally aligned (check using a level). Mark the 4 points for the holes and proceed with the drilling.
- _ Insert the relative toggle bolt nuts into the holes (use only special toggle bolts for plasterboard walls!).
- _ Tighten the 4 screws to a depth of approximately 6 mm, then secure the instrument to the wall.

WARNING! The manufacturer is not responsible for any damage caused by installation that is not performed in a proper fashion.

USING THE STEAMER FOR THE FIRST TIME

Connect the electrical power cord ② to the instrument. Make sure that the power switch ① is in the 0 position (OFF) and insert the plug into the electrical outlet. Check that the drainage screw ⑨ is inserted in the bottom of the instrument (the drainage screw is positioned transversely with respect to the drainage flow). Remove the safety cap ⑥ and fill the boiler with cold water through the filler neck. The boiler has an excess discharge tube so that it cannot be overfilled. Remove any excess water through the filler neck.

Do not fill the boiler with chemicals or other additives.

Important:

Close the filler neck by tightly screwing on the safety cap ⑥. Turn the instrument on using the power switch ① found on the left side. The heating system will turn on automatically. The red indicator light ④ will be on while the instrument is in the heating phase. After approximately 20 minutes, the Steamer X3 will reach the necessary vapor pressure and is ready for use, which is indicated by the lighting of the green light ③. The red indicator light ④ for the heater will turn off.

SWITCHING ON THE UNIT FROM COLD

In the morning, always fill the boiler before turning the instrument on.

Important:

If the unit is switched on from cold, the tank should be ventilated to prevent noises when heating up. Immediately after switching on the unit activate the switch ⑧ on the handpiece and keep pressed for 5s to ventilate the tank.

USING THE INSTRUMENT

When the green indicator lamp ④ turns on, the instrument is ready for use. To perform cleaning operations, remove the steam head ⑥ from the holder ⑦ - being careful not to touch the nozzle (risk of burning). With the steam head in a sink, turn on the switch located on the steam head ⑧ and point the jet of steam towards the object to be cleaned. When the operation is finished, put the steam head back onto its holder ⑦. The indicator light will change from green (ready for use) to red (heating) for a short period after the steam cleaning procedure.

MINIMUM WATER LEVEL

When the minimum water level in the boiler has been reached, the lower orange warning light ⑤ will turn on. The heating system will automatically shut off. Because the boiler is very well insulated, the vapor flow can be used for a certain amount

of time even after the heater has turned off. This residual vapor can be used for cleaning operations. When finished, turn off the instrument by means of the power switch and allow it to cool.

FILLING THE INSTRUMENT

Do not fill the instrument when it is still hot (risk of burning). See the safety aspects. Always use cold water to fill the Steamer X3. Once the instrument has sufficiently cooled, the safety cap ⑥ can be carefully opened.

- _ Fill Steamer X3 with cold water through the filler neck. Pour small amounts of water into the boiler until it is completely full.
- _ Close the filler neck by tightly screwing on the safety cap ⑥.
- _ Turn the instrument on by means of the power switch ①.

SAFETY ASPECTS:

WARNING! RISK OF INJURY!

The instrument is under high pressure when switched on. The safety cap ⑥ or the drainage screw ⑨ should never be opened when the instrument is on and/or hot. Never point the jet of steam at people or animals - risk of burning. Make sure that the main power switch ① is in the OFF position and that the instrument has sufficiently cooled before opening the safety cap ⑥. The instrument should no longer be pressurized when opening the safety cap. If the instrument is still under pressure when the safety cap is removed, the outcoming steam could cause

burning. If the instrument is filled when it is still hot, there is the risk that the water will be instantly vaporized and can spray out from the filler neck. Never fill steam producing devices when they are hot.

SAFETY VALVE

The Steamer X3 has a built-in safety valve that automatically opens if the pressure surpasses 6 bar. Under normal circumstances, the heater will shut off before the pressure reaches a critical level. If the safety valve activates, shut off the instrument and remove the plug from the outlet. If this occurs, the instrument should be checked by the manufacturer as a precaution.

MAINTENANCE DESCALING

The Steamer X3 must undergo a descaling procedure on a regular basis. The frequency of this procedure depends on the hardness of the water used (calcium content). Use the supplied test strips to determine the hardness of the water.

Important:

The tank should be regularly rinsed once a week, even when using water treatment/decalcification systems (salt and/or phosphate based).

MEASURING WATER HARDNESS AND DETERMINING OF DESCALING FREQUENCY

- _ Fill a container with water. Dip the test strip into the water for 1 second.
- _ Lightly shake the test strip to remove the excess water and compare it to the chromatic scale after 1 minute.
- _ **4 green areas:**
Descaling at least every 12 weeks
- _ **1 red area**
Descaling at least every 10 weeks
- _ **2 red areas:**
Descaling at least every 8 weeks
- _ **3 red areas:**
Descaling at least every 6 weeks
- _ **4 red areas:**
Descaling at least every 4 weeks

DESCALING PROCEDURE

The descaling procedure must be done when the instrument is cool and not pressurized. Do not turn the instrument on during the descaling operation! Only Kalk-X Liquid (art. 116811) must be used for this procedure.

- _ Place the Steamer X3 over a sink and open the drain screw ⑨ located at the bottom of the instrument.
- Note:** Always empty the instrument by means of the drainage screw ⑨ and not through the filling neck.
- WARNING!** Never open the drainage screw when the Steamer X3 is pressurized!
- _ Once the boiler tank been completely emptied, close the drainage screw ⑨.

- _ Pour the contents of two containers of Kalk-X liquid descaling agent (2 x 250 ml) into the boiler and then fill the tank with warm water.
- _ Allow the descaling solution to sit for at least 1 hour.
- _ After this time, reopen the drainage screw and empty the instrument.
- _ Accurately rinse the boiler tank twice using clean water to remove any calcium residues.
- _ The gasket on the safety cap ⑥ should also be checked at this time, replace it if necessary.
- _ After the descaling procedure, make sure that the drainage screw has been closed correctly.
- _ The Steamer X3 can now be refilled with water and the safety cap screwed back on. Turn the instrument on.
- _ After the warm-up period, the instrument will be ready for use.

Safety Aspects

Keep the descaling agent out of the reach of children! Avoid contact with the eyes and skin! In case of contact with the eyes, rinse with plenty of water and seek medical advice. If swallowed, seek medical advice immediately.

Do not use the descaling solution to clean the external surfaces. Only use a detergent indicated for aluminum/light metals to clean the housing.

REPLACING THE SAFETY CAP GASKETS

If the safety cap ⑥ no longer makes a good seal and/or the gasket is damaged, the gaskets must therefore be replaced.

PROCEDURE FOR REPLACING THE SAFETY CAP GASKETS

- _ Insert a flat tool underneath gasket ⑩ and lift.
- _ Cut the gasket and remove.
- _ Slip the gasket ⑩ over the threads and position it.
- _ The gasket must be completely seated and must not cover any of the screw threads.

Technical data:

D/L/H:	275 x 265 x 455 mm
Weight:	12,5 kg
Mains supply:	230 V/50 Hz
Machine rating:	1700 W
E-fuse:	T10 A
Boiler volume:	3,7 l
Vapor pressure:	4,0 bar
Warm-up time:	approx. 20 min

Accessories/Components:

Order No.:	
116811	Kalk-X Liquid (4 x 250 ml)
116814	Small sieve Ø 50 mm
116815	Large sieve Ø 75 mm
116813	Funnel
516330	Clean Steamer
516340	Stymo Net
116911	Complete safety cap
116912	Safety cap gasket
1168391	Gasket kit for steam head

Packing list:

Order No.:

116910 Steamer X3 include:

Mounting template, instruction for use, test strips for measuring water hardness, 1 x 250 ml bottle of Kalk-X descaling liquid, power cord, safety cap gasket

- _ Do not steam clean any areas or objects that conduct electricity or any live equipment.
- _ Never point the jet of steam towards people or animals.
- _ Only use distilled or tap water.
- _ Do not fill the boiler with chemicals or other additives.
- _ For descaling, use only Kalk-X Liquid (art. 116811).
- _ The tank should be regularly rinsed once a week, even when using water treatment/ decalcification systems (salt and/or phosphate based).
- _ The instrument is under high pressure when switched on. Never open the safety cap and/ or the drainage screw under these circumstances.
- _ Always make sure that the power switch ① is OFF and that the instrument is not pressurized before opening the safety cap on top of the filler neck.
- _ Fill the instrument only when it is cool.
- _ Never touch the nozzle head when the instrument is ready for use - risk of burning.
- _ **WARNING - Never open the drainage screw when the Steamer X3 is pressurized!**

All repairs and/or maintenance interventions, excluding the procedures described in the "Maintenance" paragraph must be performed by the manufacturer or by authorized technicians.

WARNINGS AND SAFETY ASPECTS:

The Steamer X3 steam cleaner was designed for dental applications. The instrument must be operated and used in accordance with the instructions for use. The manufacturer will not be responsible for any damage caused by the inappropriate and improper use of the instrument. If the intended applications for use are not mentioned in the manufacturer's instructions, the user is responsible for determining the suitability of the instrument prior to its use.

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Amann Girschbach AG
Amann Girrbach AG
Amann Girrbach GmbH